

User Manual Of Medical Face Shield

[Product Name] Medical Face Shield

[Specification And Model] L, M, S

[Product Structure And Performance]

It is composed of protective cover made of polymer material and fixing device which is not sterile and can be used once.

Main performance indexes:

The face shield surface shall be smooth, clean and free of obvious defects such as stain, rust and defect; the product shall be symmetrical structure and free of deformation. The breaking strength of the connection point between the strap and the shield shall not be less than 10N; The product should be stored in a cool and dry place, away from the water source and heating, and there should be no acid corrosion gas or other structures in the surrounding air, and no deformation. The breaking strength of the connection point between the webbing and the face shield shall not be less than 10N. The lens is transparent enough, and the normal vision can pass harmful gases.

[Application]

It is used by health-care professionals to protect their eyes and face from exposure of pathogen in body fluids, blood or the air.

[Operation Instructions]

- a. Open the package and take out the product;
- b. Put the medical isolation face shield on the face around the eyes, pull the webbing to the back of the brain, and put it on the proper position of the head
- c. After using for a period of time, if there is fog in the face shield, open the external valve for ventilation.
- d. After use, it will be discarded according to the classification of medical waste.

[Note And Caution]

- a. Carefully read the manual before use.
- b. It is limited to one-time use and shall be destroyed after use; if the initial

package is found damaged or has been opened, please do not use it.

c. The goggle lens provides limited protection. Incorrect use or failure to follow warnings and instructions can cause serious personal injury or even blindness. In order to use the product better, please read carefully

d. Although the lens has impact resistance, it is still possible to break. Dents or scratches will reduce the impact resistance and visibility of the lens. For safety reasons, the lenses must be replaced immediately in case of dents or scratches.

e. The frames and lenses of the face wear are not unbreakable. It can not provide enough protection for severe impact such as explosion, grinding wheel or grinding wheel debris. Do not wear this face shield to protect against other hazards, sports and laser radiation.

f. During wearing, avoid face injury or squeezing.

[Storage Conditions And Methods]

The face shield shall be stored in a cool and dry place away from water source and heating, and there shall be no acid corrosive gas or other harmful gas in the surrounding air. The storage time should not exceed two years from the production date.



Single Use



Do not use if package is damaged



Avoid from sunshine



Keep dry

[Term Of Validity] Two years

[Production Date / Batch No.] Refer to label on the package.